

Dermacell[®] Porous: Reopened Venous Leg Ulcer

Case Performed by: Alton Johnson, DPM

CASE STUDY

Patient & Diagnosis:

- 71-year-old male non-diabetic patient with a history of venous leg ulcer (VLU) that was successfully treated and healed 3-months prior to the current visit
- Previously healed wound was compromised due to trauma inflicted by the patient's pet, resulting in the reopening of the wound

Previous Treatment:

- Previous treatment of VLU with Grafix[®], Epifix[®], Omeza[®], Apligraf[®] and ultimately healed with TheraSkin[®]

Wound Treatment:

- Two Dermacell Porous applications following debridement
- Compression and offloading during visits every 2 weeks

Outcome:

- Wound closure following two Dermacell Porous applications in 8 weeks



Baseline wound following debridement. (Total wound size 3 cm²). First Dermacell Porous graft applied following photo documentation



At Week 4, picture depicts wound pre-debridement (30% total wound size reduction). Second Dermacell Porous applied during this visit



At Week 6, post graft application (67% total wound size reduction)



Week 8: wound closure